

Decontamination Annual Service Checklist

Autoclave

- Annual service, including fitting manufacturer's service kit spare parts
- Pressure vessel inspection to PSSR:2000
- Certification paperwork completed and issued
- Access to a senior engineer via Whatsapp video or phone support
- Takes approx. 1 hour

Validation (recommended) includes:

- Validation to HTM01-05 standard
- Run 5 or 6 cycles
- Validation paperwork issued
- Takes approx. 3-4 hours

Washer disinfecter

- Annual service, including fitting manufacturers service kit spare parts
- Certification paperwork completed and issued
- Access to a senior engineer via WhatsApp video or phone support
- Takes approx. 1 hour

Validation (recommend) includes:

- Validation to HTM01-05 standard
- Run 5 or 6 cycles
- Validation paperwork issued
- Takes approx. 3-4 hours

RO Machine

- Replacement filter sets (pre & post)
- Verification of output quality post filter replacement
- Service paperwork completed
- Visual assessment and check for leaks

 Please note, the Autoclave Annual Service, Validation and PVI will take a minimum of 5-6 hours and so will not be available to use during this time, plus 1 hour before service for cooling. Currently we can only offer 1 washer disinfecter to be completed per day.



Annual Service Checklist

Customer Site Address:

.....

.....

.....

.....

Equipment Make & Model:Serial Number:

Chair

✓	Item to check
☐	Confirm chair correctly earthed
☐	Check movement safety switches operational
☐	Fit manufacturers service kit
☐	Check incoming air and water pressures
☐	Fit service sticker and complete paperwork

Unit

✓	Item to check
☐	Check all outlets function as required
☐	Fit manufacturers service kit
☐	Clean out exhaust collection
☐	Check and adjust arm tensions, brakes and fixings
☐	Check user interface / display / membrane panel

Light

✓	Item to check
☐	Clean light front shield and reflectors
☐	Clean and test light sensor
☐	Check and adjust arm tensions, brakes and fixings

Spittoon

✓	Item to check
☐	Clean all function as required
☐	Fit manufacturers service kit
☐	Clean out suction hosing and valves
☐	Clean out spittoon drain valves
☐	Check and advise staff regarding cleaning routine

Annual Service Checklist

Customer Site Address:

.....

.....

.....

.....

Equipment Make & Model:Serial Number:

X-ray

✓	Item to check
☐	Confirm incoming voltage within acceptable range
☐	Check tensions, brakes, arms and mounting fixings are secure. Adjust as necessary
☐	Confirm earthing on machine
☐	Complete tubehead calibration (where applicable)
☐	Check and verify safety alarms, error codes and audible signals operating
☐	Complete certificate and fit sticker

Annual Service Checklist

Customer Site Address:

.....

.....

.....

.....

Equipment Make & Model:Serial Number:

Suction Pump

✓	Item to check
☐	Strip down pump
☐	Fit manufacturers service kit parts
☐	Replace Amalgam collector pot (if relevant)
☐	Clean internals of pump
☐	General external clean
☐	Reassemble, test
☐	Check displays for any error codes and confirm user interfaces are working
☐	Check and note suitability of exhaust filtration and pipework
☐	Fill in paperwork, fit service sticker

Annual Service Checklist

Customer Site Address:

.....

.....

.....

.....

Equipment Make & Model:Serial Number:

Compressor

✓	Item to check
☐	Replace inlet filters
☐	General external clean
☐	Fit manufacturers service kit, replacing parts as necessary
☐	Check for pump up time is acceptable
☐	Check and adjust pressures
☐	Drain pressure receiver
☐	If necessary, check and top-up head oil
☐	Fill in paperwork, fit service sticker